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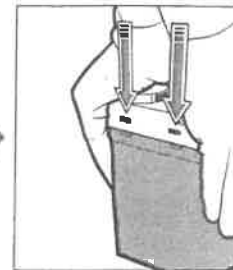
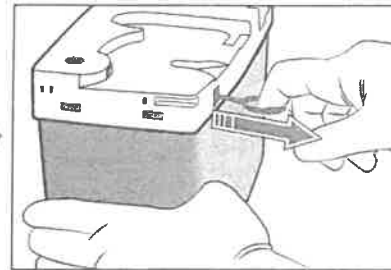
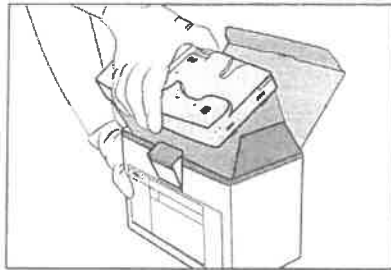
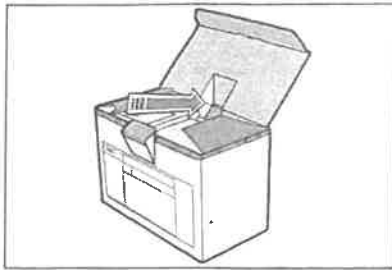
en:

Intended use:

For calibration of sensors, quality control, evaluation of accuracy and precision, rinse of measuring system and collection of waste from the ABL90 FLEX and ABL90 FLEX PLUS* analyzers.

Contents:

Biological buffers, salts, enzyme, heparin, surfactant, preservative. Contains 2-Methyl-2H-isothiazol-3-one.

Cal 1 Solution: pH: 7.30, pCO₂: 35 mmHg, pO₂: 180 mmHg, cK⁺: 4 mM, cNa⁺: 150 mM, cCa²⁺: 0.5 mM, cCl⁻: 95 mM.

Cal 2 Solution: pH: 6.8, cK⁺: 10 mM, cNa⁺: 70 mM, cCa²⁺: 2.3 mM, cCl⁻: 50 mM.

Cal 3 Solution: pCO₂: 80 mmHg, cGlu: 10 mM, cLac: 10 mM.

QC: S9030, S9040, S9050 – aqueous quality control solutions for pH, pCO₂, pO₂, cK⁺, cNa⁺, cCa²⁺, cCl⁻, cGlu, cLac, cHb, oximetry. Gas: 42.07 % O₂, 5.61 % CO₂, N₂ balance.

Chemical classification

DANGER:
H270: May cause or intensify fire; oxidizer.
P220: Keep/store away from clothing/combustible materials

Replacement procedure:

Refer to the instructions for use for your analyzer or to the instructions on the screen.

Storage:

At 2-25 °C.

On-board stability:

30 days.

Expiration date:

Refer to the label on the box.

To get the Safety Data Sheet and Certificate of Traceability, contact your local Radiometer representative.

*ABL90 FLEX PLUS is not available in USA and Canada.

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Radiometer Medical ApS
Åkandevvej 21
2700 Brønshøj
Denmark
www.radiometer.com

Radiometer, the Radiometer logo, ABL, AQT, TCM, RADIANCE, AQUIRE, PICO, CLINITUBES and QUALICHECK are trademarks of or used under license by Radiometer Medical ApS.

© 2016 Radiometer Medical ApS. All Rights Reserved.

RADIOMETER

ABL90 FLEX Sensor Cassette Pack

994-522 2014091

en: Intended use:

Sensor Cassette for the ABL90 FLEX and ABL90 FLEX PLUS* analyzers for measuring the parameters indicated on the sensor cassette.

Contents:

Sensor cassette pack with one Sensor Cassette. For the Sensor Cassette versions, refer to the instructions for use for your analyzer.

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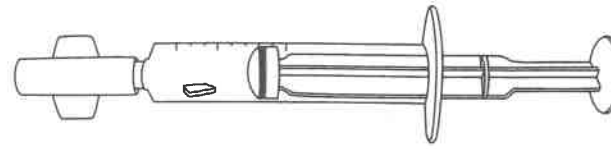
Radiometer Medical ApS
Åkandevaj 21
2700 Brønshøj
Denmark
www.radiometer.com

Radiometer, the Radiometer logo, ABL, AQT, TCM, RADIANCE, AQURE, PICO, CLINITUBES and QUALICHECK are trademarks or used under license by Radiometer Medical ApS.

© 2014 Radiometer Medical ApS. All Rights Reserved.

RADIOMETER

IVD


Rx only

STERILE EO

PICO50

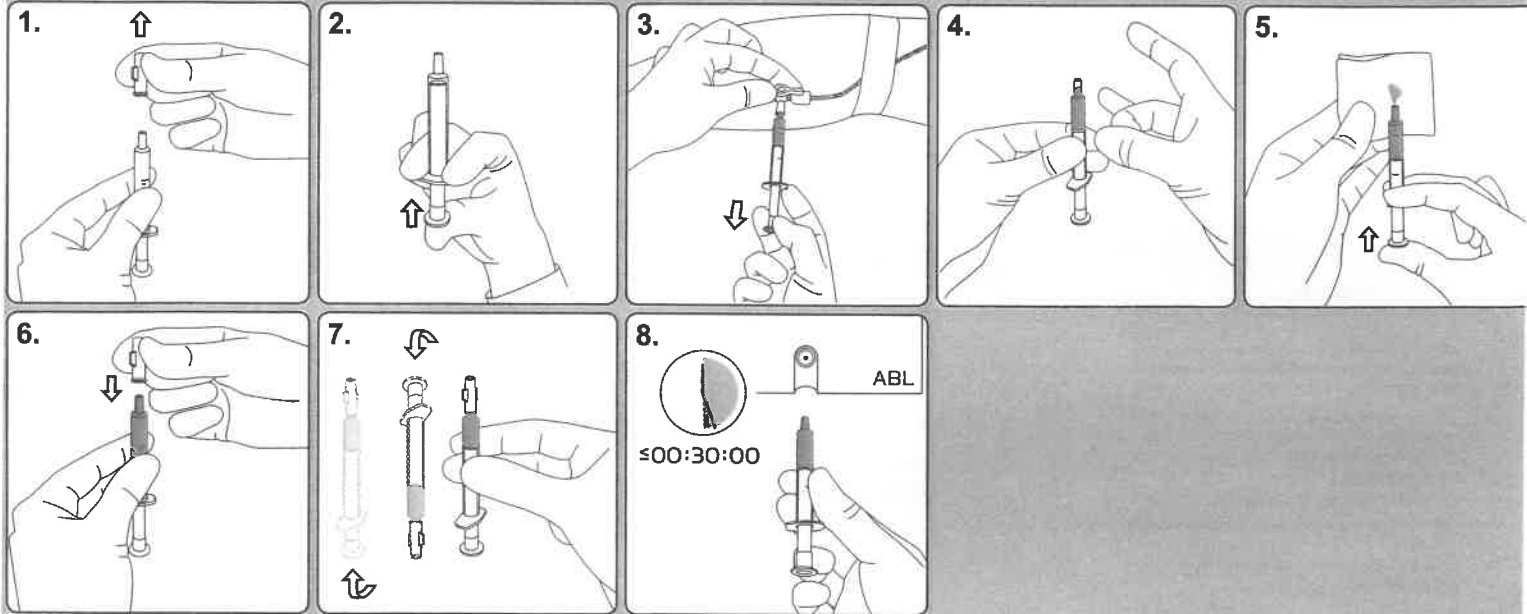0.5-2.0 ml

en

Intended use

Preheparinized, electrolyte-balanced, arterial blood sampler product line for collection of arterial samples for pH, blood gas, oximetry, electrolyte and metabolite analysis.

The sampler is preheparinized with approx. 80 IU dry electrolyte-balanced heparin (Porcine intestinal mucosa).

Not for injection.

Blood sampling should be performed by authorized personnel only.

Dispose according to local regulations/procedures.

Not made with natural rubber latex.

WARNING – Risk of infection

Always handle blood and the collection devices with care and avoid direct contact with the sample by using medical gloves.

WARNING – Risk of infection

Inspect sealed sterile package before opening. Discard sampler if seal is broken or package otherwise damaged as contents may not be sterile.

WARNING – Risk of infection

Do not point the syringe at people when expelling air. Pressing the plunger too hard may cause a sudden ejection of blood from the syringe.

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king wires & capillary caps пчета & тали за капилляри cháci drátky & čeríčky kapilár indestifter & kapillærhætter schstábchen & Kapillarschlusskappen ματα ανάμιξης και καλύμματα τριχοειδούς αγγείου irrecillos mezcladores y tapones de capilares ignetsegajad ja otsakorgid koituslangat & tulpat itateurs et bouchons de capillaire e za miješanje & čepovi kapilara verőtűskék és lezáró kupakok corette di miscelazione e tappini per capillari キングワイヤとキャピラリーキャップ ignetsegajad & otsakorgid išymo strypeliai ir kapiliaro kamšteliai ngijzertjes & afsluitdopjes indestav & Kapillærkorker tyfty mieszające i zatyczki kapilar retas de homogeneização e tampas de capilar e de amestecare și capace capilare оволочки-смесители и колпачки для капилляров šacie drótky & kapilárne čiaročky ice za mesanje & zatvaraci za kapilare ndstift & kapillärhåttor rıřtırma kabloları ve kapiller başlıklar 合丝 & 毛细管帽	Magnet Магнит Magnet Magnet Magnet Μαγνήτης Imanes Magnet Magneetti Aimant Magnet Magnes Magnete マグネット Magnetas Magnēts Magneet Magnet Magnes Íman Magnet Магнит Magnet Magnet Magnet Magnet 磁铁	Recommended for use with the following analyzers: Препоръчвани за работа със следните анализатори: Doporučeno pro uživatele přístrojové řady: Anbefalet til brug sammen med følgende apparater: Empfohlen zur Verwendung an folgenden Analysatoren: Συστήνεται για χρήση στους κάτωθι αναλυτές: Se recomienda usar con los siguientes analizadores: Soovitatakse kasutada järgmistele analüsaatoritega: Suositellaan käytettäväksi seuraavien analysaattoreiden kanssa: Utilisation recommandée avec les analyseurs suivants: Preporučuje se za upotrebu sa sljedećim analizatorima: Felhasználása a következő analizátorokhoz javasolt: Raccomandato per l'uso con i seguenti analizzatori: 次の装置での使用をお勧めします。 Rekomenduojama naudoti tokiems analizatoriams: Rekomendēts izmantošanai uz sekojošiem analizatoriem: Aanbevolen voor gebruik met de volgende analysers: Anbefalt for bruk med følgende analysatorer: Polecany do stosowania z następującymi analizatorami: Recomendado para utilização nos seguintes analisadores: Recomandat pentru utilizare cu următoarele analizoare: Рекомендуем использовать со следующими анализаторами: Doporučené pre použitie s týmito analyzátormi: Preporučuje se upotreba uz sledeće analizatore: Rekommenderad användning på följande analysatorer: ...ile kullanımı tavsiye edilir. 推荐在下列分析仪上使用:
	√ -	ABL5, ABL500 Series, ABL600 Series, ABL700 Series, ABL800 BASIC ***, ABL800 FLEX, ABL8x7 FLEX, OSM2
	√ - -	ABL90 FLEX PLUS
	- - -	ABL700 Series, ABL800 BASIC ***, ABL800 FLEX, ABL8x7 FLEX, OSM2
	-	ABL77 Series, ABL90 FLEX
	- - - -	ABL300/330, ABL500 Series, ABL 600 Series, ABL77 Series
	√ - -	ABL330, ABL5, ABL50, ABL700 Series, ABL800 BASIC ***, ABL800 FLEX, ABL90 FLEX
	√ - -	ABL30, ABL505/555, ABL600 Series, EML100/105, ABL77 Series, ABL8x7 FLEX
	-	ABL30
	√ -	ABL600 Series, ABL700 Series, ABL800 BASIC ***, ABL800 FLEX, ABL8x7 FLEX, OSM3

rada electroliticamente/et: Elektrolüütide suhtes balanseeritud/fi: Elektrolyyttitasapainotettu/fr: Equilibrée en électrolytes/hr: Elektrolit
 seerde heparine/no: Elektrolytt balansert/pl: Elektrolitowo zrównoważona/pt: Equilibrada eletroliticamente/ro: Echilibrat cu electroliti/ru:

il: Natrium/no: Natrium/pl: Sód/pt: Sódio/ ro: Sodiu/Natriu/ru: Натрий/sk: Sodík/sr: Natrijum/sv: Natrium heparin/tr: Sodyum/Natrium/zh:

i: intestinaler tierischer Mucosa/et: Ηπαρίνη από βλεννώδη εντερική μεμβράνη ζώων/es: Heparina de mucosa intestinal animal/
 vice/hu: Heparin állati bélnyálkahártyából/it: Eparina da mucosa intestinale di origine animale/ja: 獣腸粘膜から精製されたヘパリン/lt:
 z blon śluzowych jelit zwierzęcych/pt: Heparina de mucosa intestinal animal/ro: Heparină din mucoasă intestinală de origine animală/ru:
 tr: Hayvanların intestinal mukozasından alınan Heparin/zh: 肝素是从动物小肠黏膜获取的

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All rights reserved. 989-165, 2017030
Radiometer, the Radiometer logo, ABL, AQT, TCM, RADIANCE, AQURE, PICO, CLINITUBES and QUALICHECK are trademarks of or used under license by Radiometer Medical ApS.