SP-Slides

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SP-Slides

Intended use

SP-Slides are intended to be used for the automated preparation of smears from human peripheral blood samples for microscopy analysis.

SP-Slides are specifically intended to be used on the Sysmex Slide Preparation Units SP-50, SP-10, SP-100, SP-1000i and the CellaVision HemaPrep.

SP-Slides are not intended to be used for manual blood smears.

Principles of the examination method

SP-Slides are used for the preparation and staining of peripheral blood smears. The blood smear is made using the "spread" or "wedge" technique. A drop of blood is placed on the SP-Slide and this is spread rapidly by moving the spreader blade across the slide. A well-made peripheral smear is thick (red blood cells are overlapping) at the frosted end and becomes progressively thinner, with good separation of cells toward the opposite end. The so-called "zone of morphology or monolayer area" is the area of optimal thickness for light microscopy examination and should be at least 2 cm in length. The peripheral blood smear is stained so that the cytoplasmic and nuclear details of the various cell types are accentuated. Romanowsky stains or derivations thereof such as Wright, Wright-Giemsa and May-Grünwald-Giemsa are universally used in haematology.

Warnings and precautions

1. Before use, please read these Instructions for Use carefully.
2. SP-Slides are ready to use and should not be reused. Re-usage and/or improper treatment of the surfaces can lead to false results, destruction of the preparations and false diagnosis.
3. Be aware of possible harm when handling the slides and take the appropriate safety measures (e.g. wearing powder-free gloves, eye protection etc.).
4. SP-Slides may not be used when damages of the package or signs of glass breakage are present due to possible danger of injury with glass splinters.
5. When using SP-Slides with a slide preparation unit (e.g. Sysmex SP-Series), follow the manufacturer's Instructions for Use for this device.
6. After usage, SP-Slides should be regarded as potentially infectious waste. Disposal procedures should meet the requirements of the applicable local regulations.

Examination procedure

1. SP-Slides are ready to use and, after a gradual adjustment to room temperature, they can be used without further preparation.
2. If the diagnostic procedures followed by the lab make the preparation of a slide necessary, the slide should be prepared in correspondence with the instructions of the respective system and/or reagent-supplier and/or validated in-house procedures.
3. Writing fields on the SP-Slides are for the labelling of the sample and only suitable markers should be used (e.g. the label should be permanent).
4. The evaluation of the preparations on SP-Slides must be done according to the corresponding instructions of the respective system- or reagent-supplier or according to your own validated in-house procedures.

Storage and shelf life of unopened product

1. SP-Slides have to be stored in clean conditions. Storage temperature should be between 5°C and 30°C (room temperature).
2. Minimize variations in temperature and humidity during both storage and usage.
Cooling of the product can cause condensation and may lead to water forming between the slide glasses.
3. SP-Slides should be allowed to come to room temperature in the laboratory before the slide box is opened.
4. Do not use the slides after the expiration date printed on the slide box.

Manufacturer

Sysmex Europe GmbH
Bornbarch 1, 22848 Norderstedt
Germany

Product information

ZE001906 SP-Slides 30x50 slides

Date of issue or revision

11/2018