PL

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ENG

Title:
Errata for the Polish version of the user manual
1001-PMTL-270.V01

- The following changes have been made:**
- Added function for “copy basal rate”.
 - Added a slide bar scale for BG recorder.
 - Added one decimal place in bolus calculator g., page 89 IFU.
 - "low dose" "low battery" "battery exhaust" "PDA defective" such alert will be list in CPU history record.

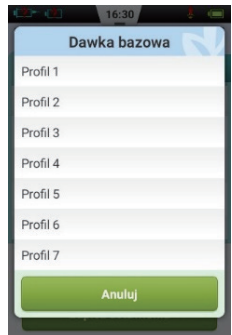
Additional functions to connect and extend:

1 Add function for “copy basal rate”

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Click the icon to choose which profile want to copy from

For example we choose Profil 3

Profil 1 will show Profil 3 data, then Confirmed the chosen copy basal rate program, then press “save setting”, copied and overwrite basal program 3 to program 1

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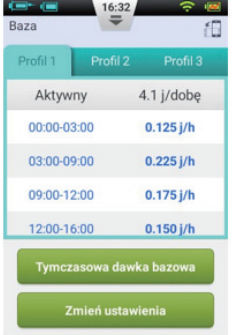
ENG

1

2 Add a slide bar scale for BG recorder

3 Add one decimal place in bolus calculator g.

4 "low dose" "low battery" "battery exhaust" "PDA system controller defective " such alert will be list in CU history record.

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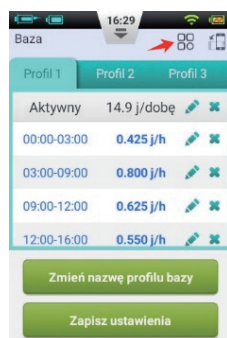
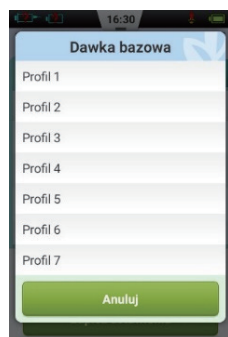
**Title:**

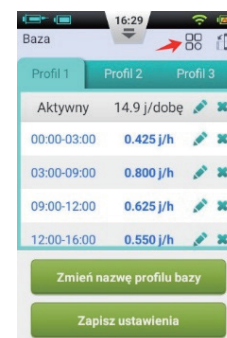
Errata for the Polish version of the user manual
1001-PMTL-270.V01

The following changes have been made:

- Added function for “copy basal rate”.
- Added a slide bar scale for BG recorder.
- Added one decimal place in bolus calculator g., page 89 IFU.
- "low dose" "low battery" "battery exhaust" "PDA defective" such alert will be list in CPU history record.

Additional functions to connect and extend:**1****Add function for “copy basal rate”**

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Click the icon to choose which profile want to copy from

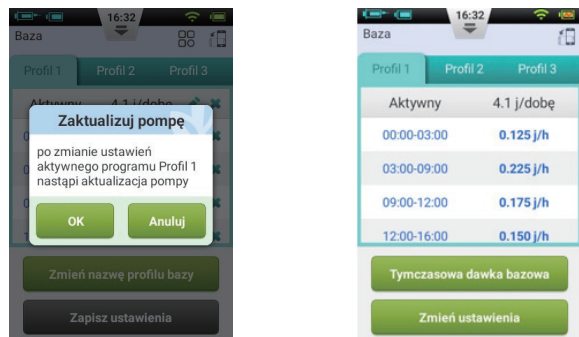
For example we choose Profil 3

Profil 1 will show Profil 3 data, then Confirmed the chosen copy basal rate program, then press “save setting”, copied and overwrite basal program 3 to program 1

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
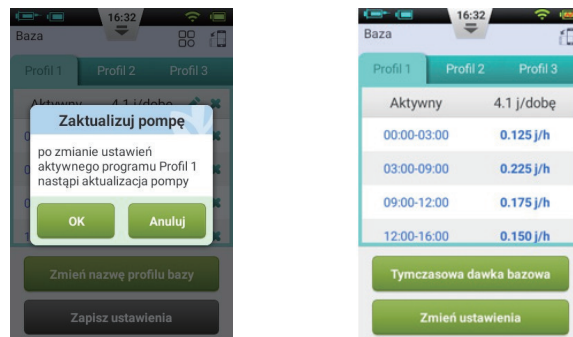
1

2

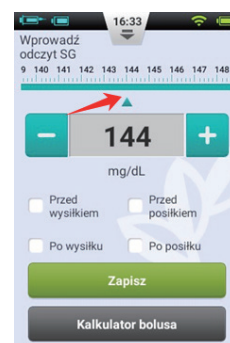Add a slide bar scale for BG recorder

3

Add one decimal place in bolus calculator g.

4

"low dose" "low battery" "battery exhaust" "PDA system controller defective " such alert will be list in CU history record.